


Conversion of .mff files using Net Station Tools

1. Locate the “.mff” files in the folder where you initially selected the path. Typically this Data will be in the EEG Folder that is on the desktop.
2. At this point, double-check each of the 4 file names ensuring the correct naming convention was followed (**subjectID_week#_condition**) and edit if necessary.
3. Open “Net Station Tools”.

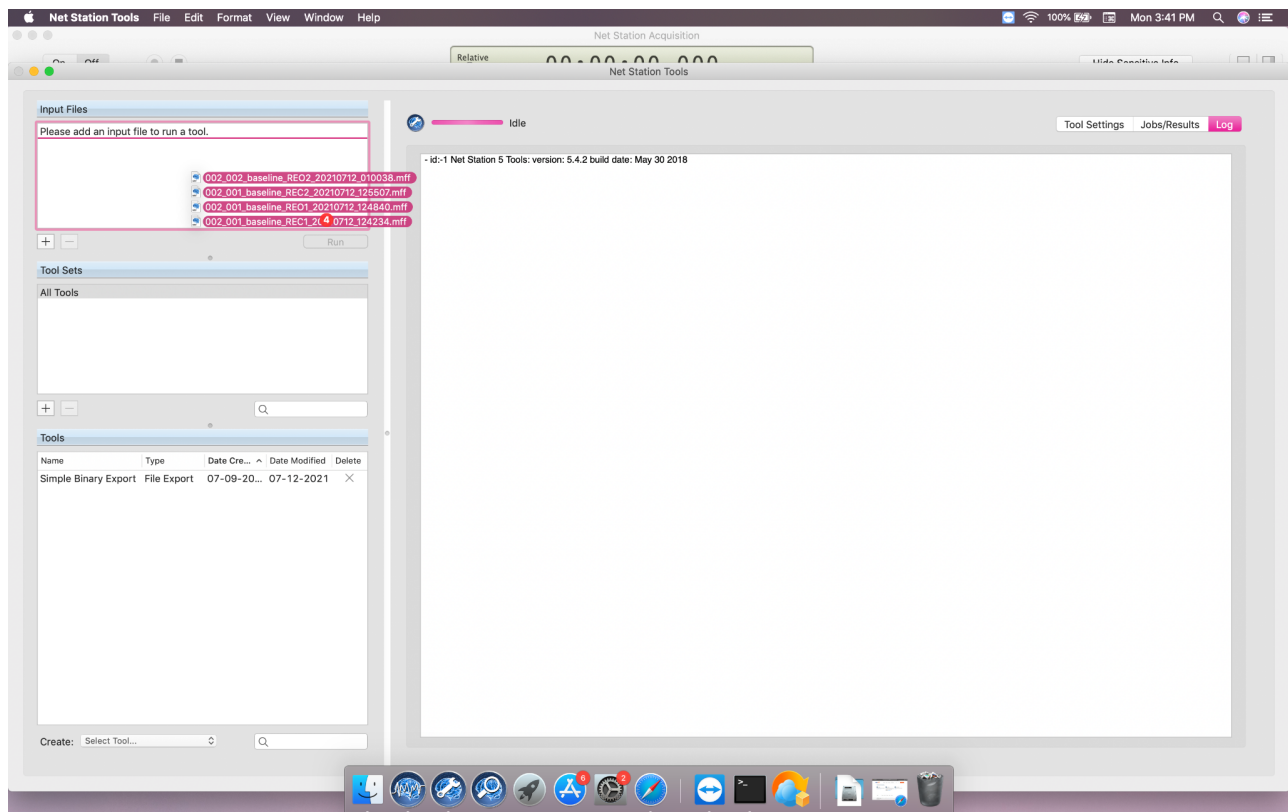

4. Click on the Simple Binary Export Tool (bottom left hand side). This would have been preset up for you.
5. Drag all four (4) .mff files into the empty box in the top right of Net Station Tools.

6. In the box below, select “Export to Binary”, then in the box below that, select “Export to Simple Binary”, then click “Run”.
 - 5 files will be exported for each .mff file.
 - .epoc, .raw, IMP.txt, GAIN.txt, and Ehst.txt
 - A total of 24 files should be added to the EEG folder (.epoc, .raw, IMP.txt, GAIN.txt, Ehst.txt and .mff) after this process is complete.

7. At the end of the conversion process, a total of 24 files should be located in the folder that contained the .mff files